



ENZYMEX L9

Enzymex L9 is a Enzymatic Disinfectant Detergent for the Pre-Disinfection of Medical Devices

Product Description

- Enzymex L9 is an enzymatic detergent for the pre-disinfection of medical devices in an immersion bath.
- Enzymex L9 is a complex trienzymatic (lipase, amylase, and protease) combined with effective surfactants
- Instrument integrity guaranteed for 72 hours in immersion-bath
- Proven efficiency on biofilm
- Tuberculoidal activity

Application

- Enzymex L9 can be used in the operating theatre, for endoscopy, ophthalmology, and contact lenses
- Can be used in an ultrasonic vat

Composition

- Non-ionic surfactants
- Quarternary ammonium
- Enzymes: lipase, amylase, and protease
- Sequestrants

Characteristics

- Form: Green liquid
- pH at 0.5%: 7.5
- Expiry date: 2 years after manufacture

Precautions for Use

See Safety Data Sheet

Storage Conditions

- Storage between 5-25°C
- Keep container tightly closed and in a well-ventilated place.
- Keep cool.
- Protect from sunlight.
- Store in original container and in an upright position.
- Keep away from food, drink, and animal feed.

Material Compatibility

Metal Stainless Steel

Plastics PSU, PVC, PE

Other Materials Ceramics

Advice Application as intended will not harm materials.
See the product's technical sheet

Hazardous product. Follow precautions of use. Read label and product information before use. Medical device class IIb.

Instructions for Use

Preparation of the pre-cleaning bath:

- Wear suitable personal protective equipment (see Safety Data Sheet)
- Dilute the appropriate volume of **ENZYMEX L9** in 5L of water and homogenize the solution (see dilution table)

Examples for the preparation of 5 litres of working solution

Concentration Desired	Concentrate Per 5-litres Water	Volume Prepared
0.05%	25 ml	5-litre
2.0%	100 ml	5-litre
5%	250 ml	5-litre

Use:

1. Soiled instruments have to be fully immersed.
 2. Scrub medical devices and use a swab if necessary.
 3. Contact time: according to the antimicrobial activity desired (see disinfecting properties)
 4. Rinse thoroughly
- Renew the bath for each use.*

Certification


Enzymex L9 is a class IIb medical device and is CE marked according to Directive 93/42/EC



0459

Expert Opinions and Further Information

- Enzymex L9 has been assessed by independent laboratories.
- For further details on expert opinions, more information on the product or individual questions contact us by e-mail (info@airande.global) or telephone (+61 2 9844 5846).
- Airandé is certified by BSI to ISO 13485
- Franklab has a manufacturing certificate under GMP and is certified EN ISO 13485 certificate.

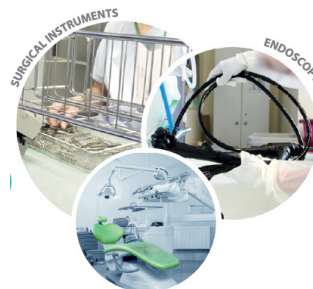

Packaging and Ordering

Single Pack	Units Per Carton	Product Number
25 ml	250	AIR_10-43547
1-litre bottle	12	AIR_10-43516
5-litre canister	4	AIR_10-43520



Distributed Globally by Airandé Pty Limited
Level 13, Suite 1A, 465 Victori Avenue,
CHATSWOOD NSW 2067 AUSTRALIA
Tel: +61 2 9844 Fax: +61 2 9844
E: sales@airande.global
W: www.airande.global

Business Center - Cercle du Lac
Rue de Rodeuhaie, 1
1348 LOUVAIN-LA-NEUVE BELGIUM
E: info.europe@airande.global



Manufactured by Franklab
3 av. des Frères 8180 MONTIGNY LE BRETONNEUX FRANCE
Tel: +33 (0) 1 39 44 93 40 Fax: +33 (0) 1 39 44 93 41
www.franklab.fr



ENZYMEX L9

Enzymex L9 is a Enzymatic Disinfectant Detergent for the Pre-Disinfection of Medical Devices

Disinfecting Properties

Standard	Strain	Concentration	Contact Time
Bactericidal			
EN 13727	<i>S. aureus, P. aeruginosa, E. hirae</i>	0.5%	5 min
EN 14561	<i>S. aureus, P. aeruginosa, E. hirae</i>	0.5%	10 min
Yeasts			
EN 13624	<i>C. albicans</i>	0.5%	5 min
EN 14562	<i>C. albicans</i>	0.5%	10 min
Viruses			
EN 14476 + A2	<i>Herpes Virus</i>	0.5%	5 min
EN 14476 + A1	<i>DBV Virus, Hepatitis C model, HIV Virus</i>	0.5%	10 min
EN 17111	<i>Vaccinia Virus</i>	0.5%	10 min
Tuberculocidal			
EN 14348 : 2005	<i>Mycobacterium terrae</i>	5%	10 min
		2%	60 min
NF EN 14563 : 2009	<i>Mycobacterium terrae</i>	5%	10 min
		2%	60 min



1-litre



250 ml



5-litre



Distributed Globally by Airandé Pty Limited
 Level 13, Suite 1A, 465 Victori Avenue,
 CHATSWOOD NSW 2067 AUSTRALIA
 Tel: +61 2 9844 Fax: +61 2 9844
 E: sales@airandé.global
 W: www.airandé.global

Business Center - Cercle du Lac
 Rue de Rodeuhaie, 1
 1348 LOUVAIN-LA-NEUVE BELGIUM
 E: info.europe@airandé.global



Manufactured by Franklab
 3 av. des Frènes 8180 MONTIGNY LE BRETONNEUX FRANCE
 Tel: +33 (0) 1 39 44 93 40 Fax: +33 (0) 1 39 44 93 41
 www.franklab.fr

Technical Sheet_V1_042023

