



PLIWA Löffelreiniger

Concentrate for preparation of a working solution for cleaning of dental impression trays

Product Description

- PLIWA Löffelreiniger is a concentrate for preparation of a working solution for the cleaning of dental impression trays

Application

- PLIWA Löffelreiniger is suitable for cleaning stainless steel or plastic dental impression trays

Special Instructions

- Read safety datasheet before use.
- Close container immediately after use.
- The product does not have any corrosive or decomposition effects that remove material residues.
- Safety data sheet available at www.airande.global

Shelf Life and Expiry Date

- The shelf life of the working solution of PLIWA Löffelreiniger is 1 day.
- Use concentrate within 12 months after opening
- See product label for expiry date.
- Do not use the product after the expiry date.

Storage Advice

- Storage between 5-25°C
- Keep container tightly closed and in a well-ventilated place.
- Keep away from food, drink, and animal feed.
- Protect from sunlight.
- Store in original container and in an upright position.
- See safety datasheet for further advice.

Instructions For Use

Put on protective clothing / gloves and, if necessary, eye protection before cleaning.

1. Prepare working solution. 5-10% depending on the degree of soiling. Example for a 10% working solution: Mix 900 ml drinking water (max. 30°C) + 100 ml concentrate.
2. Clean dental impression trays roughly from attaching material (alginates, cement, silicone) and soak them in the working solution if possible over night.
3. Remove the dental impression trays from the working solution and remove residues of material with a brush
4. Rinse dental impression trays under running water.

Labelling according to regulation (EC) No. 1272/2008 (CLP)

For information on labelling according to regulation (EC) No 1272/2008 (CLP) see safety datasheet and label



Labelling according to regulation 648/2004/EC on detergents

- ≥5% - <15% non-ionic surfactants
- < 5% anionic surfactants
- disinfectants, preservations agents
- (METHYLISOTHIAZOLINONE, BENZISOTHIAZOLINONE)

Packaging and Ordering

Single Package	Units Per Carton	Product Number
1-litre cylinder bottle	10	AIR_10-200
5-litre canister	1	AIR_10-203
Accessories		
Dosing pump		AIR_50006
Drainage tap		AIR_50009



Distributed Globally by Airandé Pty Limited
 Level 13, Suite 1A, 465 Victori Avenue,
 CHATSWOOD NSW 2067 AUSTRALIA
 Tel: +61 2 9844 Fax: +61 2 9844
 E: sales@airande.global
 W: www.airande.global

Business Center - Cercle du Lac
 Rue de Rodeuhaie, 1
 1348 LOUVAIN-LA-NEUVE BELGIUM
 E: info.europe@airande.global



Manufactured by PLIWA Hygiene GmbH
 Pliwastraße 2 34323 MALSFELD GERMANY
 Tel: +49 (0) 5661 73 17 0 Fax: +49 (0) 5661 73 17 10
www.pliwa.de



PLIWA Löffelreiniger

Concentrate for preparation of a working solution for cleaning of dental impression trays

Further Information

For further information on the product or individual questions contact us by e-mail (info@airande.global) or telephone (+61 2 9844 5846).

Data sheet according to regulation (EC) 648/2004 Annex VII Section D

Name of Product: Pliwa Löffelreiniger

Manufacturer: PLIWA Hygien GmbH, Pliwastr, 2, 34323 Malsfeld Germany

Ingredients:

AQUA
ISOPROPYL ALCOHOL
2-Propylheptanoethoxilate
PROPYLENE GLYCOL
SODIUM COCETH-30 SULFATE
GLYCOLIC ACID
BENZOTRIAZOLE



Distributed Globally by Airandé Pty Limited
Level 13, Suite 1A, 465 Victori Avenue,
CHATSWOOD NSW 2067 AUSTRALIA
Tel: +61 2 9844 Fax: +61 2 9844
E: sales@airande.global
W: www.airande.global

Business Center - Cercle du Lac
Rue de Rodeuhaie, 1
1348 LOUVAIN-LA-NEUVE BELGIUM
E: info.europe@airande.global

Manufactured by PLIWA Hygiene GmbH
Pliwastraße 2 34323 MALSFELD GERMANY
Tel: +49 (0) 5661 73 17 0 Fax: +49 (0) 5661 73 17 10
www.pliwa.de

