



Ecoseptol Foam

Disinfectant Detergent Foam Alcohol Free for Medical Devices



airandé
SUPERIOR DISINFECTION SOLUTIONS

Distributed Globally by Airande Pty Limited
Level 13, Suite 1A, 465 Victori Avenue,
CHATSWOOD NSW 2067 AUSTRALIA
Tel: +61 2 9844 Fax: +61 2 9844
E: sales@airande.global
W: www.airande.global

Business Center - Cercle du Lac
Rue de Rodeuhaie, 1
1348 LOUVAIN-LA-NEUVE BELGIUM
E: info.europe@airande.global



Franklab
notre expertise l'Ultra-Propreté

Manufactured by Franklab
3 av. des Frères 8180 MONTIGNY LE BRETONNEUX FRANCE
Tel: +33 (0) 1 39 44 93 40 Fax: +33 (0) 1 39 44 93 41
www.franklab.fr



Ecoseptol Foam

Disinfectant Detergent Foam Alcohol Free for Medical Devices

Product Description

- Ecoseptol Foam is a alcohol free disinfectant detergent foam.
- Neutral pH solution.
- Chloride-free formulation.
- Active on enveloped viruses.
- Ready-to-use.
- Bottle with foam dispenser.

Composition

- Cationic surfactants.
- Non-ionic surfactants.
- Chlorinexihidine digluconate.
- Sequestrants.
- Excipients.

Characteristics

- Appearance: Clear Flowable liquid
- pH: 6

Shelf Life

- 3 years from date of manufacture.

Instructions For Use

1st Use of spray gun:

- Wear gloves and unlock the spray.

Use (usage):

- Spray Ecoseptol Foam directly on the surface or on a dry single use wipe.
- Spread the solution with the wipe.
- Contact time: according to the antimicrobial activity desired (see disinfecting properties).
- Do not rinse.
*The product does not require rinsing, unless specified by the manufacturer's instructions or when in contact with skin or mucous membranes.
- Appropriately dispose of the single use wipe.
- Repeat whenever necessary.

Further Information

- Ecoseptol Foam has been assessed by independent laboratories.
- For further details on expert opinions, more information on the product or individual questions contact us by e-mail (info@airande.global) or telephone (+61 2 9844 5826).
- Airandé is certified by BSI to ISO 13485.
- Franklab has a manufacturing certification to ISO 9001: 2015 and to EN ISO 13485.

Special Advice & Precautions for Use

- See Safety Data Sheet

Disinfecting Properties

Activity	Standards	Contact Time
Bactericidal	EN 13727 + A2/EN 13697/EN 1276	1 min
	EN 14561	5 min
Yeasticidal	EN 13624/EN 1650 +A1	1 min
	EN 13697/EN 14562	5 min
Fungicidal	EN 13697	5 min
	EN 1650/EN 13624/EN14562	10 min
Virucidal	EN 14476 + A1 <i>Rotavirus, PRV (hepatitis B model virus), DBV (hepatitis C model virus) VRS Virus</i>	5 min
	EN 14476 +A2 <i>Vaccinia Virus</i>	5 min
Sporicidal	NFT 72-230/EN 13704 <i>B. subtilis and C difficile</i>	15 min

Packaging and Ordering

Single Pack	Units Per Carton	Product Number
750ml bottle - with foam trigger dispenser	12	AIR_14-146538



Distributed Globally by Airandé Pty Limited
Level 13, Suite 1A, 465 Victori Avenue,
CHATSWOOD NSW 2067 AUSTRALIA
Tel: +61 2 9844 Fax: +61 2 9844
E: sales@airande.global
W: www.airande.global

Business Center - Cercle du Lac
Rue de Rodeuhaie, 1
1348 LOUVAIN-LA-NEUVE BELGIUM
E: info.europe@airande.global



Manufactured by Franklab
3 av. des Frères 8180 MONTIGNY LE BRETONNEUX FRANCE
Tel: +33 (0) 1 39 44 93 40 Fax: +33 (0) 1 39 44 93 41
www.franklab.fr

Technical Sheet_V1_042023

