



# FB SPRAY

## Disinfectant Detergent for High Touch Surfaces



FOOD CONTACT



**airandé**  
SUPERIOR DISINFECTION SOLUTIONS

Distributed Globally by Airandé Pty Limited  
Level 13, Suite 1A, 465 Victori Avenue,  
CHATSWOOD NSW 2067 AUSTRALIA  
Tel: +61 2 9844 Fax: +61 2 9844  
E: sales@airandé.global  
W: www.airandé.global

Business Center - Cercle du Lac  
Rue de Rodeuhaie, 1  
1348 LOUVAIN-LA-NEUVE BELGIUM  
E: info.europe@airandé.global

**Fb Franklab®**  
notre expertise l'Ultra-Propreté

Manufactured by Franklab  
3 av. des Frères 8180 MONTIGNY LE BRETONNEUX FRANCE  
Tel: +33 (0) 1 39 44 93 40 Fax: +33 (0) 1 39 44 93 41  
www.franklab.fr

Technical Sheet\_V1\_042023



# FB SPRAY

## DISINFECTANT DETERGENT FOR HIGH-TOUCH SURFACES

### Product Description

- FB Spray detergent is easy to use and is ideally suited for the disinfecting of high-touch and high-traffic areas.
- It is a neutral pH solution that is active and effective on enveloped viruses.
- It is suitable for biocleaning of surfaces in contact with food-stuffs.
- It is compatible with a wide range of materials.
- It is registered as a biocide: TP 02/03/04 (inventory N°: 65065)
- Active on *Bacillus subtilis* and *Clostridium difficile* spores.
- FB Spray is free of *Carcinogen Mutagen substance toxic for Reproduction (CMR)*, color, and fragrance.

### Composition

- Non-ionic surfactants
- Quaternary ammonium

### Characteristics

- FORM: Flowable liquid
- pH: 6±1

### Shelf Life

- 3 years from date of manufacture.

### Instructions For Use

#### First Use of the Wipes

- Wear gloves and unlock the spray.

#### Use

1. Spray the **FB Spray** uniformly on the surface or on a dry single use wipe. cessary.
2. Spread the solution with the wipe.
3. Surface must be wet for a minimum of 1 minute.
4. Leave on to dry or wipe with a dry single use wipe.

*FB Spray does not require rinsing unless otherwise specified by the manufacturer of surfaces or for surfaces in contact with foodstuffs.*

### Further Information

- FB Wipes have been assessed by independent laboratories.
- For further details on expert opinions, more information on the product or individual questions contact us by e-mail (info@airande.global) or telephone (+61 2 9844 5846).
- Airandé is certified by BSI to ISO 13485
- Franklab has a manufacturing certification to ISO 9001: 2015 and to EN ISO 13485.

### Special Advice & Precautions for Use

- See Safety Data Sheet

### Disinfecting Properties

Activity	Standards	Contact Time
Bactericidal	EN 13727 + A2/EN 13697/EN 1276	1 min
	EN 14561	5 min
Yeasticidal	EN 13624/EN 1650 +A1	1 min
	EN 13697/EN 14562	5 min
Fungicidal	EN 13697	5 min
	EN 1650/EN 13624/EN14562	10 min
Virucidal	EN 14476 + A1 <i>Rotavirus, PRV (hepatitis B model virus), DBV (hepatitis C model virus) VRS Virus</i>	5 min
	EN 14476 +A2 <i>Vaccinia Virus</i>	5 min
Sporicidal	NFT 72-230/EN 13704 <i>B. subtilis and C difficile</i>	15 min

### Packaging and Ordering

Single Pack	Units Per Carton	Product Number
750ml bottle	12	AIR_10-1078640
5-litre canister	4	AIR_10-1078620



Distributed Globally by Airande Pty Limited  
Level 13, Suite 1A, 465 Victori Avenue,  
CHATSWOOD NSW 2067 AUSTRALIA  
Tel: +61 2 9844 Fax: +61 2 9844  
E: sales@airande.global  
W: www.airande.global

Business Center - Cercle du Lac  
Rue de Rodeuhaie, 1  
1348 LOUVAIN-LA-NEUVE BELGIUM  
E: info.europe@airande.global



Manufactured by Franklab  
3 av. des Frères 8180 MONTIGNY LE BRETONNEUX FRANCE  
Tel: +33 (0) 1 39 44 93 40 Fax: +33 (0) 1 39 44 93 41  
www.franklab.fr

