



FB WIPES

Cleaning and Disinfecting Wipes



Distributed Globally by Airandé Pty Limited
Level 13, Suite 1A, 465 Victori Avenue,
CHATSWOOD NSW 2067 AUSTRALIA
Tel: +61 2 9844 Fax: +61 2 9844
E: sales@airandé.global
W: www.airandé.global

Business Center - Cercle du Lac
Rue de Rodeuhaie, 1
1348 LOUVAIN-LA-NEUVE BELGIUM
E: info.europe@airandé.global

Manufactured by Franklab
3 av. des Frères 8180 MONTIGNY LE BRETONNEUX FRANCE
Tel: +33 (0) 1 39 44 93 40 Fax: +33 (0) 1 39 44 93 41
www.franklab.fr





FB WIPES

Cleaning and Disinfecting Wipes

Product Description

- FB Wipes are cleaning and disinfecting wipes whose purpose is the treatment of surfaces and non-invasive medical devices and to control microbiological risks linked to the environment.
- Ready to use impregnated wipes adapted to large surfaces (FB wipes XXL).
- impregnated with a pH neutral, fragrance-free solution.
- Effective against enveloped viruses.
- Active on *Bacillus subtilis* and *Clostridium difficile* spores.
- Compatible with a wide range of materials including synthetic coatings (see below).
- Compatible with food usage.

Application

- FB Wipes are suitable for cleaning wipeable surfaces including but not limited to patient beds, operating tables, monitors, haemodialysis equipment, respirators, and dental units.

Composition

- Non-ionic surfactants
- Quaternary ammonium

Characteristics

- FB Wipes XXL: 240 x 300 mm)
- pH of the impregnating solution: 6

Shelf Life

- 3 years from date of manufacture - once opened, the optimal shelf life is maintained **only if the pack is closed tightly** after use

See product label for expiry date. Do not use the product after expiry date.

Instructions For Use

First Use of the Wipes

- *Wear gloves and remove the protective lid.*
- Take out the 1st wipe with one hand **while holding the pack with the other hand.**

Use

1. Clean the area by using as many wipes as necessary.
2. Ensure that the contact time is consistent with the desired antimicrobial activity.
3. Close the pack tightly after use. Do not reuse the wipe.

Does not require rinsing unless otherwise specified by the manufacturer of the treated surface coating or for surfaces in contact with foodstuffs.

Materials Compatibility

Stainless Steel, Aluminium, Chrome, Copper, Brass, Corian, HDPE, Plexiglass, Polypropylene, PVC, Neoprene, Silicone, Latex, Nitrile, Austenitic stainless steel, Martensitic stainless steel, Glass, Epoxy resin glue, Rubber

Storage Conditions

- Keep container tightly closed and in a well-ventilated place.
- Protect from sunlight.
- Only store in original packaging made of an identical material to the original.
- For further information on storage of the product, see label and safety data sheet.
- Storage temperature: 5-25°C

Further Information

- FB Wipes have been assessed by independent laboratories.
- For further details on expert opinions, more information on the product or individual questions contact us by e-mail (info@airande.global) or telephone (+61 2 9844 5846).
- Airandé is certified by BSI to ISO 13485
- Franklab has a manufacturing certification to ISO 9001: 2015 and to EN ISO 13485.

Special Advice

- Always wash hands after handling.
- Avoid contact with other chemicals
- Read the label, product information and safety datasheet before use.
- Application by qualified personnel only.
- Never eat, drink or smoke during use.
- Remove and wash contaminated clothing before re-using.
- Ensure that there is adequate ventilation, especially in confined areas.

Antimicrobial Activity

Activity	Standards	Contact Time
Bactericidal	EN 1276/EN 13727/ EN 13697	1 min
	EN 16615	2 min
	EN 14561	5 min
Yeasticidal	EN 13624/EN 1650	1 min
	EN 16615	2 min
	EN 13697/EN 14562	5 min
Fungicidal	EN 13697	5 min
	EN 1650/EN 13624/ EN14562	10 min
Virucidal	EN 14476 <i>Rotavirus, PRV (hepatitis B model virus), DBV (hepatitis C model virus) VRS Virus</i>	5 min
	EN 14476 <i>Vaccinia Virus</i>	5 min
Sporicidal	EN 13704 <i>B. subtilis and C difficile</i>	15 min

Packaging and Ordering

Single Pack	Units Per Carton	Product Number
Pack of 60 Wipes (240 x 300 mm)	12	AIR_10-10785142



Distributed Globally by Airandé Pty Limited
Level 13, Suite 1A, 465 Victoria Avenue,
CHATSWOOD NSW 2067 AUSTRALIA
Tel: +61 2 9844 Fax: +61 2 9844
E: sales@airande.global
W: www.airande.global

Business Center - Cercle du Lac
Rue de Rodeuhaie, 1
1348 LOUVAIN-LA-NEUVE BELGIUM
E: info.europe@airande.global



Manufactured by Franklab
3 av. des Frères 8180 MONTIGNY LE BRETONNEUX FRANCE
Tel: +33 (0) 1 39 44 93 40 Fax: +33 (0) 1 39 44 93 41
www.franklab.fr