



PLIWA Hygientücher Lemon Fresh AF

Alcohol-Soaked Disinfection Wipes

Product Description

- PLIWA Hygientücher Lemon Fresh AF-Wipes are ready-to-use alcohol-soaked disinfection wipes (using the soaking solution PLIWA Lemon Fresh AF containing alcohol and quaternary ammonium compounds) for application with a dispenser can.

Application

- PLIWA Hygientücher Lemon Fresh AF Wipes are suitable for rapid disinfection of alcohol-resistant, wipeable surfaces in medical facilities e.g., dental and medical practices, aged care homes, etc.
- Suitable for use on a broad range of materials (see below).
- The wipes are for single use only.

Product Composition

- 100 g ready to use solution contains: 33.66 g ethanol, 15.84 g 2-propanol, 0.05 g didecyldimethyl-ammonium chloride, 70% perfume* (not applicable for perfume-free)

* Please note that perfume may cause allergies

Contra-indications

- Not tested for food and veterinary areas
- Not suitable for personal body hygiene, disinfection of skin and hands and disinfection of medical devices.

Special Advice

- Use biocidal products safely.
- Read the label, product information and safety datasheet before use.
- Application by qualified personnel only.
- Keep out of the reach of children.
- Do not soak with other chemicals

Shelf Life

- The shelf life of PLIWA Hygientücher Lemon Fresh AF Wipes is indicated on the container/label of the refill bags.
- Do not use after the expiry date.

Expert Opinions and Further Information

- PLIWA Hygientücher Lemon Fresh AF Wipes have been assessed by independent laboratories.
- To exclude adverse effects on the skin upon unintentional contact with the product, a dermatological survey was carried out by the company, Dermatest.
- For further details on expert opinions, more information on the product or individual questions contact us by e-mail (info@airande.global) or telephone (+61 2 9844 5846).
- Airandé is certified by BSI to ISO 13485
- PLIWA Hygiene GmbH has a manufacturing authorisation as per §13 German medicines act as well as a GMP- and an EN ISO 13485 certificate.

Storage Conditions

- Keep container tightly closed and in a well-ventilated place.
- Keep cool.
- Protect from sunlight.
- Keep away from sources of ignition.
- Store in original container and in an upright position.
- For further information on storage of the product, see label and safety data sheet.

Characteristics

- Appearance: clear and colourless
- pH value: 5.0-6.0
- Density at 20°C: 0.900 - 0.920
- Solubility in water: miscible

Material Compatibility

Metal	Aluminium, Stainless Steel, Copper, Brass
Plastics	PSU, PVC, Silicone
Rubber	EPDM, Latex
Advice	Application as intended will not harm materials. See the product's technical sheet



Manufactured By
PLIWA Hygiene GmbH
Pliwa-Str.2
D-34323 Malsfeld-Ostheim
T: 0049 56 61/73 17-0



airandé
SUPERIOR DISINFECTION SOLUTIONS

Distributed By
Airande Pty Limited
Suite 1a, Level 13
465 Victoria Avenue
Chatswood NSW 2067 Australia
T: +61 2 9844 5826 E: info@airande.global



PLIWA Hygientücher Lemon Fresh AF

Alcohol-Soaked Disinfection Wipes

Packaging and Ordering

Single Pack	Units Per Carton	Product Number
Refill bag		
Lemon Fresh - 120 Wipes in refill bag	12 x 120 wipes/refill bag	AIR_10-116
Lemon Fresh - 200 Wipes in refill bag	6 x 200 wipes/refill bag	AIR_10-140
Dispenser cans incl refill bag		
Lemon Fresh- 120 wipes in refill/bag	30 x 120 wipes/refill bag	AIR 10-119
Lemon Fresh - 200 wipes in refill bag	15 x 200 wipes/refill bag	AIR 10-141
Related Products		
Product	Description	Packaging Size
PLIWA Lemon Fresh AF	<ul style="list-style-type: none"> Ready to Use alcoholic solution for surface disinfection, medical device Lemon Fresh, without perfum, 	<ul style="list-style-type: none"> 1-litre 5-litre
Quick & Clean S Wipes	<ul style="list-style-type: none"> Ready to use alcohol-soaked disinfection wipes in a refill bag for application with a dispenser bucket, medical device Lemon Fresh, without perfume 	<ul style="list-style-type: none"> 70 wipes 90 wipes
Manufacturing Partner		

PLIWA Hygiene GmbH



Manufactured By
 PLIWA Hygiene GmbH
 Pliwa-Str.2
 D-34323 Malsfeld-Ostheim
 T: 0049 56 6173 17-0



airandé
 SUPERIOR DISINFECTION SOLUTIONS

Distributed By
 Airande Pty Limited
 Suite 1a, Level 13
 465 Victoria Avenue
 Chatswood NSW 2067 Australia
 T: +61 2 9844 5826 E: info@airande.global



PLIWA Hygientücher Lemon Fresh AF

Alcohol-Soaked Disinfection Wipes

Antimicrobial Activity (ready to use product, at room temperature)

Standard	Strain	Conditions	Contact Time
Bactericidal			
EN 13727	<i>P. aeruginosa, S. aureus, E. hirae E. coli, MRSA</i>	Dirty	30 sec
EN 16615	<i>S. aureus</i>	Dirty	5 min
EN 13697	<i>S. aureus, E. hirae, P. aeruginosa, e.coli</i>	Dirty	5 min
Mycobacteria/TB			
EN 14348	<i>M. terrae, M. avium</i>	Dirty	1 min
Yeasts			
EN 13624	<i>C. albicans</i>	Dirty	30 sec
EN 13697	<i>C. albicans</i>	Dirty	5 min
Viruses			
Enveloped Viruses			
EN 14476	MVA Modified Vaccinavirus Ankara	Dirty	30 sec
	BVDV		
Non-enveloped viruses			
EN14476	Adenovirus Type 5	Dirty	5 min
	Human Rotavirus	Clean	
	MNV murine Norovirus	Dirty	
EN 16777	MNV murine Norovirus	Dirty	
	Adenovirus Type 5	Dirty	



Manufactured By
PLIWA Hygiene GmbH
Pliwa-Str.2
D-34323 Malsfeld-Ostheim
T: 0049 56 61/73 17-0



Distributed By
Airande Pty Limited
Suite 1a, Level 13
465 Victoria Avenue
Chatswood NSW 2067 Australia
T: +61 2 9844 5826 E: info@airande.global

