



PLIWA Quick and Clean S-wipes

Alcohol-Soaked Disinfection Wipes

Product Description

- PLIWA Quick & Clean S-wipes are ready-to-use alcohol-soaked disinfection wipes in a refill bag for application with a dispenser bucket.
- They are soaked with PLIWA Lemon Fresh AF (containing alcohol and quaternary ammonium compounds) and can be used directly after insertion into the dispenser bucket
- PLIWA Quick and Clean S-Wipes provide a universal and time-saving application
- PLIWA Quick and Clean S-Wipes are intended for single use only

Application

- PLIWA Quick & Clean S-Wipes are suitable for rapid disinfection of alcohol-resistant surfaces of non-invasive, non-critical medical devices, medical equipment, as well as all kinds of wipeable surfaces
- Suitable for:
 1. surfaces close to patients, dental chairs, examination couches, operating tables
 2. areas in medical and dental practices which are used for medical purposes
 3. rapid disinfection between surgeries and procedures
 4. clothes such as OR shoes, OR aprons, X-ray aprons
 5. Alcohol-resistant surfaces of medical equipment
 6. Materials according to material compatibility (for further information see the product's technical sheet)

Product Composition

- 100 g ready to use solution contains: 33.66 g ethanol, 15.84 g 2-propanol, 0.5 g didecylmethyl-ammonium chloride, 70% perfume* (not applicable for perfume-free)

* Please note that perfume may cause allergies

Contra-indications

- Not tested for food and veterinary areas
- Not suitable for disinfection of invasive medical devices
- Not for use for disinfection of semi-critical and critical medical devices

Shelf Life

- The shelf life of PLIWA Quick & Clean S-Wipes is indicated on the container/label. Shelf-life of the ready-to-use wipes in the dispenser bucket: 28 days from the date of opening months after opening if the lid is closed properly and tightly every time a wipe is taken out.

Identification, Certificates

Medical device class IIa / MD 0108

Certificate of efficacy issued by VAH (Association of Applied Hygiene)

Expert Opinions and Further Information

- PLIWA Quick & Clean S-Wipes have been assessed by independent laboratories.
- To exclude adverse effects on the skin upon unintentional contact with the product, a dermatological survey was carried out by the company, Dermatest.
- For further details on expert opinions, more information on the product or individual questions contact us by e-mail (info@airande.global) or telephone (+61 2 9844 5846).
- Airandé is certified by BSI to ISO 13485
- PLIWA Hygiene GmbH has a manufacturing authorisation as per §13 German medicines act as well as a GMP- and an EN ISO 13485 certificate.

Storage Conditions

- Keep container tightly closed and in a well-ventilated place.
- Keep cool.
- Protect from sunlight.
- Keep away from sources of ignition.
- Store in original container and in an upright position.
- Keep away from food, drink, and animal feed.

Characteristics

- Appearance: clear and colourless liquid
- pH value: 5.0-6.0
- Density at 20°C: 0.900 - 0.920
- Solubility in water: miscible
- Flashpoint (EN 22719): 22.75 + 0.5°C
- See safety data sheet for further advice.

Material Compatibility

Metal	Aluminium, Stainless Steel, Copper, Brass
Plastics	PSU, PVC, Silicone
Rubber	EPDM, Latex
Advice	Application as intended will not harm materials. See the product's technical sheet



Manufactured By
PLIWA Hygiene GmbH
Pliwa-Str.2
D-34323 Malsfeld-Ostheim
T: 0049 56 61/73 17-0



airandé
SUPERIOR DISINFECTION SOLUTIONS

Distributed By
Airande Pty Limited
Suite 1a, Level 13
465 Victoria Avenue
Chatswood NSW 2067 Australia
T: +61 2 9844 5826 E: info@airande.global



PLIWA Quick and Clean S-Wipes

Alcohol-Soaked Disinfection Wipes

Packaging and Ordering

Unit	Units Per Carton	Product Number
Lemon Fresh		
Lemon Fresh - 70 Wipes in refill bag	2	AIR_10-452
Lemon Fresh - 90 Wipes in refill bag	2	AIR_10-453
No Perfume		
No Perfume: 70 Wipes in refill bag	2	AIR_10-552
No Perfume: 90 Wipes in refill bag	2	AIR_10-553
Accessories		
Dispenser bucket	1	AIR_93308

Manufacturing Partner

PLIWA Hygiene GmbH



Manufactured By
PLIWA Hygiene GmbH
Pliwa-Str.2
D-34323 Malsfeld-Ostheim
T: 0049 56 61/73 17-0



airandé
SUPERIOR DISINFECTION SOLUTIONS

Distributed By
Airande Pty Limited
Suite 1a, Level 13
465 Victoria Avenue
Chatswood NSW 2067 Australia
T: +61 2 9844 5826 E: info@airande.global



PLIWA Quick and Clean S-Wipes

Alcohol-Soaked Disinfection Wipes

Antimicrobial Activity (ready to use product, at room temperature)

Standard	Strain	Conditions	Contact Time
Bactericidal			
EN 13727	<i>P. aeruginosa, S. aureus, E. hirae</i> <i>E. coli, MRSA</i>	Dirty	30 sec
EN 16615	<i>P. aeruginosa, S. aureus, E. hirae</i>	Dirty	5 min
VAH (Standard Methods)**	<i>S. aureus, E. hirae, P. aeruginosa</i>	Dirty	5 min
Mycobacteria/TB			
EN 14348	<i>M. terrae, M. avium</i>	Dirty	1 min
Yeasts			
EN 13624	<i>C. albicans</i>	Dirty	30 sec
EN 16615	<i>C. albicans</i>	Dirty	5 min
VAH (Standard Methods)**	<i>C. albicans</i>	Dirty	5 min
Viruses			
Enveloped Viruses			
EN 14476	<i>MVA Modified Vaccinavirus</i> <i>Ankara</i>	Dirty	30 sec
	<i>BVDV</i>		
Non-enveloped viruses			
EN14476	<i>Adenovirus Type 5</i>	Dirty	5 min
	<i>Human Rotavirus</i>	Clean	
	<i>MNV murine Norovirus</i>	Dirty	
EN 16777	<i>MNV murine Norovirus</i>	Dirty	
	<i>Adenovirus Type 5</i>	Dirty	



Manufactured By
 PLIWA Hygiene GmbH
 Pliwa-Str.2
 D-34323 Malsfeld-Ostheim
 T: 0049 56 61/73 17-0



Distributed By
 Airande Pty Limited
 Suite 1a, Level 13
 465 Victoria Avenue
 Chatswood NSW 2067 Australia
 T: +61 2 9844 5826 E: info@airande.global

